


# 幽門桿菌抗原快速檢驗試劑

# 檢體採集說明書


## STEP 1

標示檢體相關訊息於藍色檢體處理管上。

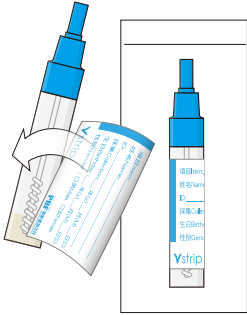


## STEP 2

打開檢體處理管瓶蓋。將附於蓋上之採檢棒插入糞便檢體以取得少量糞便(如圖示)避免取量過多,以免造成無效結果或帶狀棕色線的出現。

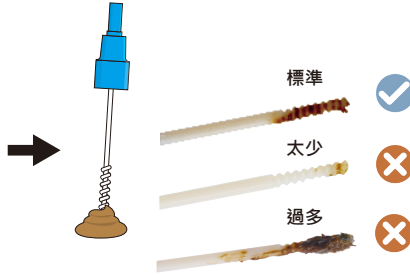

## STEP 3

將採檢棒放回管中,旋緊瓶蓋。反轉、搖動或震盪檢體處理管,使稀釋液與糞便檢體徹底混合均勻。請立即將檢體處理管送回檢驗單位。

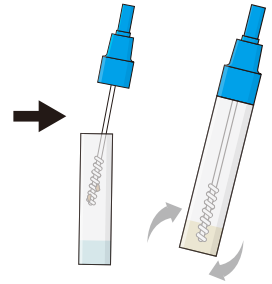


# H. pylori Antigen Rapid Test

# SPECIMEN PREPARATION


## STEP 1

Label the sample diluent tube with a patient or other control information.

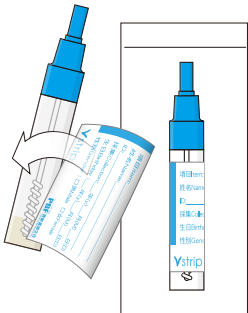

## STEP 2

Unscrew the sample diluent- tubecap, use the sample stick (attached to the cap) to transfer a small piece of stool (like picture) into the stool collection tubes. Avoid excessive specimen sample on sample stick to ensure valid interpretation of the result.

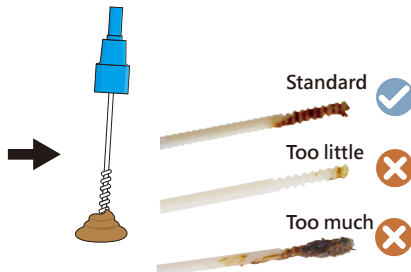

## STEP 3

Close the tube and shake the tube in order to assure good sample dispersion.

